

Kit Contents:

1) Vacuum

2) Power Cable (x2)

3) Power Cable Bag

4) Carry Strap

Vacuum User Interface / Features:

A) Battery Level Indicator

B) Power Switch

C) AC Adapter Port

D) Exhaust Vent (under flap)

E) Vacuum Hose Port

F) Vacuum Port Cap

G) Internal Access Zipper

H) Accessory Loops

Warnings:

- When using the SafER Negative Pressure System, Healthcare workers and others must use proper PPE when exposed to a patient with suspected or confirmed COVID-19 or other sources of SARS-CoV-2 (See OSHA's PPE standards at 29 CFR 1910 Subpart I).
- Do not allow fluid or debris to be suctioned.
- Monitor for hypoxia or hypercapnea.



SafER Vacuum Instructions For Use

Caution! Do not turn on vacuum without a connected filter and hose assembly!

- 1.) Remove vacuum port cover (F)
- 2.) Insert approved SafER filter hose assembly to vacuum port (E)
- 3.) Toggle rocker switch (B) to the on position to power vacuum
- 4.) Perform negative pressure procedure
- 5.) Toggle rocker switch to the off position to turn vacuum off

SafER Vacuum Cleaning Instructions

- 1.) Wipe external surfaces of vacuum bag with germicidal wipe
- 2.) For more aggressive cleaning use mild detergent on a damp soft cloth
- 3.) Do not over moisten cloth
- 4.) Do not allow liquid in / around charging port
- 5.) Dry sides with soft cloth

SafER Vacuum Charging Instructions

- 1.) Locate charging port (C).
- 2.) Plug charger (round barrel connector) into charging port
- 3.) Plug the other end of charger into 110VAC outlet
- 4.) Charger light will turn green when fully charged

Note: Charge time from empty battery to full is approx. 2 hours.

SafER Vacuum Charge Indicator

- 1.) Push the on button (A) to turn the meter on
- 2.) The battery meter is displayed in five bars and respectfully represents the current battery capacity in the battery
- 3.) Push the button again to turn the meter off

