

AC/DC PORTABLE VACUUM INSTRUCTIONS FOR USE

ATTENTION!

Read all precautions and warnings carefully before attempting to operate. Retain for future reference. Please visit: www.safermedical.com/instructions for latest version before use.



Scan For
Latest
Instructions
For Use

WARNING!

- When using the Safer Negative Pressure System, Healthcare workers and others must use proper PPE when exposed to a patient with suspected or confirmed COVID-19 or other sources of SARS-CoV-2 (See OSHA's PPE standards at 29 CFR 1910 Subpart I).
- Do not allow fluid or debris to be suctioned.
- Monitor for hypoxia or hypercapnea.

DANGER!

Never operate this unit when flammable materials or vapors are present because electrical devices produce arcs or sparks that can cause a fire or explosion.

IMPORTANT SAFETY INSTRUCTIONS

When using an electrical appliance, basic precautions should always be followed, including the following: **READ ALL INSTRUCTIONS BEFORE USING THIS APPLIANCE.** If the Safer AC/DC Portable Vacuum is used in a manner not specified in this manual, the protection provided by the equipment may be impaired.

WARNING!

TO REDUCE THE RISK OF FIRE, ELECTRIC SHOCK OR INJURY:

1. Connect to a properly grounded outlet only. See Grounding Instructions.
2. Do not expose to rain. Only store indoors.
3. Do not allow to be used as a toy. Close attention is necessary when used near children.
4. Use only as described in this manual. Use only Manufacturer's recommended attachments.
5. Do not use with damaged cord or plug. If appliance is not working as it should, has been dropped, damaged, left outdoors or dropped into water, contact your Sales Representative.
6. Do not pull or carry by cord, use cord as a handle, close a door on cord or pull cord around sharp edges or corners. Do not run appliance over cord. Keep cord away from heated surfaces.
7. Do not unplug by pulling on cord. To unplug, grasp the plug; not the cord.

8. Do not handle plug or appliance with wet hands.
9. Do not leave the cord lying on the floor. It can become a tripping hazard.
10. Do not put any object into openings. Do not use with any openings blocked; keep free of dust, lint, hair and anything that may reduce air flow.
11. Keep hair, loose clothing, fingers and all parts of body away from openings and moving parts.
12. Do not use without filters in place.
13. Do not use to pick up flammable or combustible liquids such as gasoline or use in areas where they may be present.
14. STAY ALERT. Watch what you are doing and use common sense. Do not use vacuum cleaner when you are tired, distracted or under the influence of drugs, alcohol or medication causing diminished control.

SAVE THESE INSTRUCTIONS

GROUNDING INSTRUCTIONS

This appliance must be grounded. If it should malfunction or breakdown, grounding provides a path of least resistance for electric current to reduce the risk of electric shock. The plug must be inserted into an appropriate outlet that is properly installed and grounded in accordance with all local codes and ordinances.

WARNING!

Improper connection of the equipment-grounding conductor can result in a risk of electric shock. Check with a qualified electrician or service person if you are in doubt as to whether the outlet is properly grounded. Do not modify the plug provided with the appliance – if it will not fit the outlet, have a proper outlet installed by a qualified electrician. Rated supply voltage: 29.4V DC (CV), Current: 4A (CC: 3.6A-4.4A), Wattage: 105-130W

Kit Contents:

- 1) Vacuum
- 2) Carry Strap
- 3) 110v Power Supply | Battery Charger

Vacuum User Interface / Features:

- A) Battery Level Indicator
- B) Power Switch
- C) Power Supply | Charger Input Port
- D) Exhaust Vent (under flap)
- E) Vacuum Hose Port
- F) Vacuum Port Cap
- G) Internal Access Zipper
- H) Accessory Loops



SafER Vacuum Instructions For Use

Caution! Do not turn on vacuum without a connected filter and hose assembly!

- 1) Remove vacuum port cover (F).
- 2) Insert approved SafER filter hose assembly to vacuum port (E).
- 3) Toggle rocker switch (B) to the on position to power vacuum. Light will illuminate when switched on and battery level sufficient to power equipment.

Note: if switch is on, and no light illuminated, the battery needs charging.

- 4) Perform negative pressure procedure.
- 5) Toggle rocker switch to the off position to turn vacuum off, light will turn off.

SafER Vacuum Cleaning Instructions

- 1) Wipe external surfaces of vacuum bag with germicidal wipe.
- 2) For more aggressive cleaning use mild detergent on a damp soft cloth.
- 3) Do not over moisten cloth.
- 4) Do not allow liquid in / around charging port.
- 5) Dry sides with soft cloth.

SafER Vacuum Charging Instructions


- 1) Locate power supply | charger input port (C).
- 2) Plug charger (round barrel connector) into charging port
- 3) Plug the other end of charger into 110VAC outlet
- 4) Power Supply will illuminate RED LED when charging
- 5) Charger light will turn green when fully charged

Note: Charge time from empty battery to full is approx. 2 hours.
If using the equipment while charging, more time will be required.

SafER Vacuum Charge Indicator

- 1) Push the on button (A) to turn the meter on
- 2) The battery meter is displayed in five bars and respectfully represents the current battery capacity in the battery
- 3) Push the button again to turn the meter off

Each full LED represents the approximate battery charge. each light bar level is 20% of charge. solid light bar = 20%, flashing bar = 10%

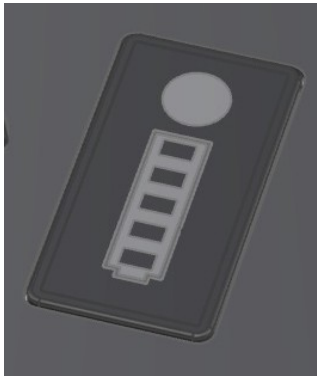

- 10% = 1 led flashing
- 20% = 1 led solid
- 30% = 1 led solid + 1 led flashing
- 40% = 2 led solid
- 50% = 2 led solid + 1 led flashing
- 60% = 3 led solid
- 70% = 3 led solid + 1 led flashing
- 80% = 4 led solid
- 90% = 4 led solid + 1 led flashing
- 100% = 5 led solid

